28181
ELCROSS[®] Ankle

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Intended use/Indications

This product should be used to unload weak ankles and as support for light sprains and sports injuries.

Contraindications

Ligament ruptures, fractures, severe oedema from diabetes, or other circulation problems.

Adjustments

This product must only be altered by a medical professional person.

Warnings

Make sure that the support is not tightened to hard as this can cause circulation problems. If any skin irritation occurs, immediately remove the support. Check skin for pressure marks on a daily basis.

Material specification

This product is made from cotton with an elastic strap. This product is not flame retardant.





Wash

Close all Velcro straps into material prior to laundering. Use a washing bag. Machine wash in 40°C. Dry on a flat surface. Do not tumble dry or dry clean.

Contact


If any problem's occurs, remove the support and contact your physician or were the product was purchased.

Instruction

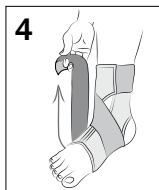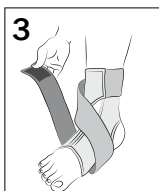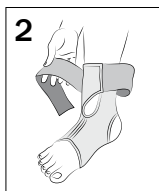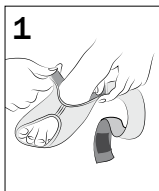Check that the support is complete and without damage.

1. Pull on to the foot like a sock.
2. The elastic strap should first go behind the leg and then over the foot.
3. Pull the strap under the foot and give it a slight pull before it comes up on top of the foot again (like a figure-8).
4. To increase the stability, make a slightly harder pull to lift the lateral edge to prevent the foot from falling outwards.
5. Adjust for comfort.

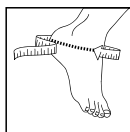Art.nr.	Size	Circumferens anke
28181 1011	Left, Small	20 - 25 cm
28181 2011	Right, Small	20 - 25 cm
28181 1013	Left, Large	25 - 30 cm
28181 2013	Right, Large	25 - 30 cm

 Camp Scandinavia AB, Karbingatan 38
 SE-254 67 Helsingborg, Sweden
 Phone +46 42 25 27 01

SVERIGE	DENMARK	SUOMI	NORGE	UK/IRELAND	USA	INTERNATIONAL
Tel 042-25 27 01	43 96 66 99	09 350 76 30	23 23 31 20	+353 42 932 8177	888-678 6548	+46 42 25 27 01
Fax 042-25 27 25	43 43 22 66	09 350 76 388	23 23 31 21	+353 42 932 8182	800-289 0809	+46 42 25 27 25
info@camp.se	info@camp.dk	info@camp.fi	info@camp.no	info@allarduk.co.uk	info@allardusa.com	info@allardint.com

Support For Better Life • www.allardsupport.com