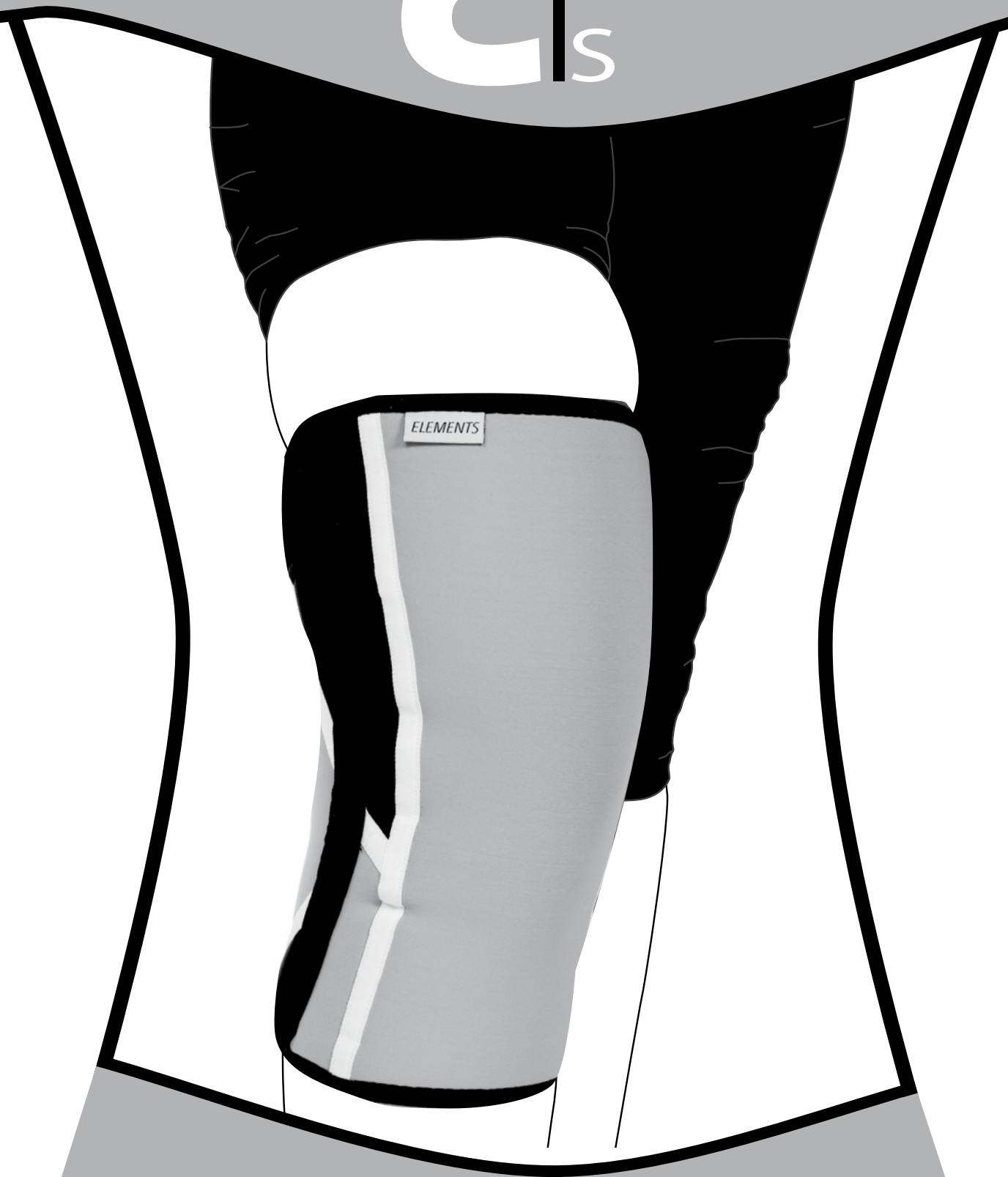
C | S

ELEMENTS

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ELEMENTS KNEE SUPPORT

Carefully read this instruction sheet before use!

Indications / Intended Use

This product is intended for treatment of mild and temporary conditions such as pain and inflammation. The support gives a light compression and keeps the knee warm. This support is only intended for one person use.

Contraindications

Edema, Severe swelling, Ligament injuries, Leg ulcers.

Adjustments

This product should only be altered by a medical professional person.

Warnings

Check that you have the correct size. Some conditions can result in swelling that can mean variations in size. If this is the first time you are using a support it is important to start slowly. Use the support for a short period of time the first day, then increase the use gradually. The support should not be used overnight. Do not use directly on open wounds. Use only on dry skin. Do not use lotion on your skin. Make sure the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from: Polyamid, polyurethane and spandex with metal stays. This product does not contain Latex, and is not flame retardant.

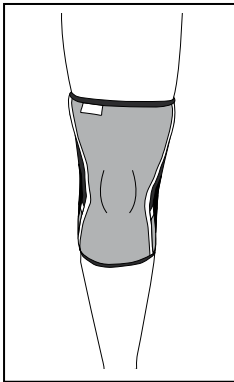
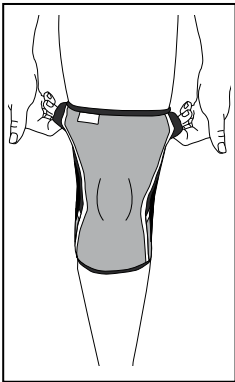
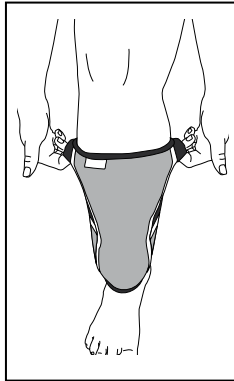
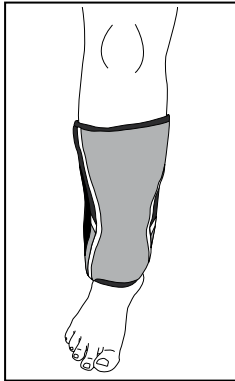
Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for Use

1. Check that the support is complete and without damage.
2. Put the foot through the support.
3. Pull the support gently by using the two small pockets on the side.
4. Make sure the support is positioned correctly and comfortably over the knee.

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Camp Scandinavia AB, Karbingatan 38