ATTO

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movinglife.com

VÄLKOMMEN TILL ATTO!

VÄLKOMMEN TILL ATTO!

INLEDNING

Viktig notis

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Figure 4

3. Press the central release button until it clicks and remains pressed (see Figure 5).

Figure 5

4. Press the rod button and tilt it away from the main ATTO body (see Figure 6).

Figure 6

5. Hold ATTO with two hands (see Figure 7).

Figure 7

6. Tilt ATTO toward the rod until the front wheel touches the ground (see Figure 8).

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Figure 8

7. Using the main handle, pull the main body away from the rod, thus rolling the back wheels away from the front wheel (see Figure 9).

Figure 9

8. Gently move the handle down until ATTO is flat as shown in Figure 9 (see Figure 10).

NOTE Keep holding the rod at upright position

Figure 10

9. Place handle in the designated recess until it clicks and locks in place (see Figure 11).

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Figure 11

10. Lock the rod in upright position by pushing the front lever down until it clicks and locks in place (see Figure 12). . In case the front lever require too much force to operate, you should apply horizontal pressure on the main handle towards the front of the scooter. This will release the pressure on the front lever and will make it easier to open.

Figure 12

11. Place one hand on the tiller and with the other hand release the secure lever of the back handle and bring the handle to upright position at 90 degrees (see Figure 13).

NOTE For proper operation please do not flip the handle to 180 degrees from closed position. If you did and you are unable to lift the chair backwards, return the handle to its original position and try again.

Figure 13

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Using the handle, pull the back of the chair backwards until it clicks and locks in place (see Figure 14).

Figure 14

12. Continue using the handle for pulling up the chair until it clicks and locks in place (see Figure 15).

WARNING Do not sit on the chair before it clicks in place to prevent it from sliding down again.

Figure 15

13. Open both sides of the handle bar (see Figure 17).

Figure 16

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Adjusting for maximum comfort

Several adjustments can be made for maximum comfort. Some adjustments can be made by the user, while other adjustments **must only be made by a certified technician**. The adjustments that can be made by the user are detailed in this section.

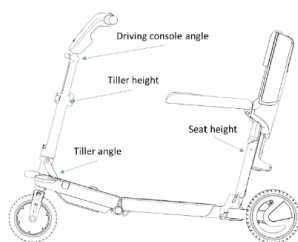
Figure 17 Comfort adjustments

PINCH POINT

Operations described in this chapter are performed near and around pinching points. Be careful and follow instructions to prevent injury. Make sure your fingers are away from moving parts that may be closing gaps while moving.

Adjusting Driving Console height

Use the rod lever to adjust the height of the handle bar at your convenience. Bring the lever back to the secure position once you are done (see Figure 17).

Figure 18 Driving Console height lever

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Adjusting Driving Console angle

ATTO's Driving Console can be adjusted to your comfort in a variety of angles. To adjust the Driving Console you should first be seated on your mobility scooter:

1. Hold the Driving Console in one hand and release the adjusting lever using the other hand (see Figure 19).
2. Adjust the Driving Console angle so you can reach and hold it comfortably with a clear view of the display area.
3. Tighten the adjustment lever when you are done.

Figure 19 Driving Console tilt lever

Seat height adjustment

REQUIRES TOOLS

Adjusting seat height must be performed only by certified technician to prevent injury and damage.

Although ATTO's seat is thin and stylish, it was designed to provide comfort and safety and can be adjusted according to user's preferences. The seat mechanism can be locked at one of three fixed heights. To adjust seat height please follow the following steps, no tools needed:

1. ATTO has to be in opened position in order to adjust seat height. Lift the seat to upright position, locate the seat height adjustment cover at the seat base right under the seat pivots and remove it (see Figure 20).

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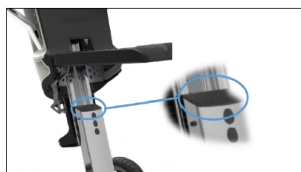
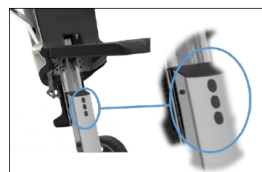
2. Figure 20 Seat height cover

3. Under the cover you will find a metal bracket that has to be removed. Hold the bracket between your fingers and lift it out. Remember the bracket position to make it easier to put it back in place again.
4. Once the bracket has been removed you should be able to remove the bulky metal pin (located in one of the three adjustment holes) holding the seat in its place. Remove the pin while holding the seat with your other hand (see Figure 21).

5. Figure 21 Seat height adjustment holes

6. Now you can adjust the seat to any of the three holes height and reinsert the pin to hold it in place.
7. Reinsert the metal bracket and the cover.

WARNING When the seat is in its highest position, the scooter will be more inclined to tip over.

Tiller angle adjustment

REQUIRES TOOLS

The tiller angle adjustment must be done by certified technician only, to prevent injury and damage.

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The Steering Rod (tiller) angle can be adjusted in one of two positions according to user comfort. The default position will be adequate for most users, larger users may want the Steering Rod further away from their body for more space. In order to adjust the angle you need a 4 mm Allen wrench (hex key).

1. ATTO has to be in the opened position in order to adjust the tiller angle. Once opened, locate and release the two Allen (hex) screws shown in Figure 22 while holding the tiller in upright position (you need to hold the tiller to prevent it from falling while releasing the screws).
2. Move the tiller into the desired position and tighten the two screws back again using a thread locker (such as Loctite).

Figure 22 Steering Rod angle adjustment screws

Driving Console

The Driving Console (your scooter's dashboard) contains all the controls and display needed for operating your mobility scooter while driving.

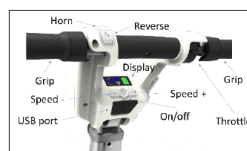
Figure 23 Driving Console overview

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ON/OFF and speed control buttons

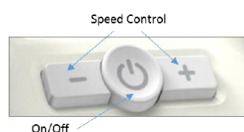
Figure 24 On/Off and speed control

- A. The On/Off button is located at the center of the Driving Console below the display panel. To turn ATTO on press and hold the button until ATTO starts.
- B. The two buttons on the sides of the ON/OFF switch marked with "+" and "-" are your speed limit control. Along with the number shown on the display panel, they allow you to set the maximum speed ATTO will develop at full throttle. While ATTO is not moving, Click the "+" button to increase the speed and the "-" button to decrease the speed.

WARNING When driving in crowded places and near obstacles, use extra care and reduce the maximum speed to a limit allowing you to control your mobility scooter without presenting a risk to other people and to yourself.

Display panel

Figure 25 Display Panel

Battery level meter – When ATTO power is ON the battery meter indicates the approximate battery charge level. As with all Li-ion batteries, the charge level indication is accurate only when the battery is fully charged. All other indications are approximate and intended to give you a rough idea of the battery charge level. Please refer to the "Specification" chapter in this

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Gasreglage

Owner's Manual for more information on your battery range.

NOTE Your scooter will automatically turn itself off if the battery output falls below operation voltage.

Direction mode – ATTO can drive both forward and backward. The current driving direction is shown on the display panel by an arrow pointing forward or backward, depending on user's selection. To switch the driving direction press on the direction button located near the left hand grip.

Throttle

Figure 26 Driving Throttle

The throttle is used for driving and controlling the driving speed. Place your thumb on the throttle lever and press it down to accelerate. To decelerate, release the throttle lever to your level of comfort. To bring the scooter to a complete stop simply remove your thumb from the lever which will set the lever to its "off" position, bringing the scooter to a halt after 1.2 meter.

Reverse and horn buttons

Figure 27 Reverse and horn buttons

- A. ATTO is equipped with a driving direction button. Press on the button to change the driving direction. Once the driving direction has

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changed, ATTO will sound a single beep when in forward driving mode, and a double beep when in reverse driving mode. In addition, a large arrow on the display panel will indicate the current driving mode. When ATTO is turned on it is always in Forward driving mode.

- B. Press the horn button to sound the horn. Always use the horn when needed to prevent accident or injury.

⚠ WARNING Always bring the mobility scooter to a full stop before changing the driving direction.

USB port

ATTO is equipped with a general USB charging port providing up to 1.2A charging current. If an error occurred, the port is disabled and the scooter will operate as normal. The error will be reset after restarting of the scooter's controller (power off/on cycle).

Display Dimming

Pressing the '+' and '-' buttons simultaneously will toggle the dimming option.

Bee sounds

Beeps caused by pressing buttons may be enabled or disabled. To Enable, press the '+' and 'Horn' buttons simultaneously. To Mute the beeps, press the '-' and 'Horn' buttons simultaneously (horn is always active).

Handle bar grips

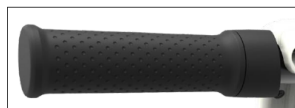
Figure 28 Handlebar grip

There are two grips on the handlebar, one on each side.

To change the grips from folded position to drive position simply lift the grip around its pivot and it will

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Driving

NOTE

INSPECTIONS PRIOR TO USING YOUR SCOOTER:

1. Please check that the battery is in its place and verify it is locked and not protruding off the surface.
2. Please check that the battery is fully charged or has enough capacity to carry you to your desired destination.
3. Verify that the freewheel lever is in locked position.
4. Verify that all locking parts are in fully opened position and locked in place.

Turning ATTO on

Press and hold the "On" button to turn ATTO on. All lights will come on momentarily and then the following information will be indicated:

- Set maximum speed
- Forward driving direction (default)
- Battery charge level status

See Figure 29 Typical display after turning on.

Figure 29 Typical display after turning on

- We recommend that you select a maximum speed setting (using the "+" and "-" buttons near the On/Off switch) at which you feel comfortable, safe, and in control of your scooter.
- Make sure that you are seated safely and properly on your ATTO.
- For your first driving sessions, make sure that your ATTO starts out on a level surface and that you will continue driving on a level surface.

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- When driving down a ramp, keep ATTO's speed set to the slowest speed setting to ensure a safely controlled descent.
- Avoid sudden stops and starts.

When climbing an incline, try to keep ATTO moving. If you must stop, start up again slowly, and then accelerate cautiously. When driving down an incline, do so by adjustment the speed to the slowest setting and driving in the forward direction only. If your scooter starts to move down the incline faster than you anticipated or desired, allow it to come to a complete stop by releasing the throttle. Then push the throttle forward slightly to ensure a safely controlled descent.

⚠ WARNING When climbing an incline, do not zigzag or drive at an angle up the face of the incline. Drive your ATTO straight up the incline. This greatly reduces the possibility tipping over or falling. Always exercise extreme caution when negotiating an incline.

⚠ WARNING Do not drive your ATTO across the side of an incline or diagonally up or down an incline; Avoid stopping, as much as possible, while driving up or down an incline.

⚠ WARNING You should not travel up or down a potentially hazardous incline (i.e., areas covered with snow, ice, cut grass, or wet leaves).

⚠ WARNING When on any sort of an incline or decline, never place the ATTO in freewheel mode while seated on it or standing next to it.

⚠ WARNING Even though your ATTO is capable of climbing slopes greater than that illustrated in Figure 30, do not, under any circumstances, exceed the incline guidelines or any other specifications presented in this manual. Doing so could cause your

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Turning corners

Turning at high speed may cause a tipping over. Factors affecting the possibility of tipping over include turning speed, steering angle (how sharply you are turning the steering tiller), uneven road surfaces, inclined road surfaces, a transition from low grip area to high grip area (as moving from a grass surface to a paved surface - especially at high speed while turning), and abrupt changes of direction. High speed turning is not recommended. If you feel you are about to tip over while turning, reduce speed and steering angle (i.e., make the turn wider) to prevent your scooter from tipping over.

WARNING When you turn a sharp corner, reduce speed. When you use the scooter at a high speed, do not turn sharply. This will reduce the risk of overturning and falling. Always exercise discretion when taking a turn to avoid injury and damage to property.

Folding (closing)

1. Make sure ATTO is on flat ground before folding it.

PINCH POINT Operations described in this chapter are done near and around pinching points. Be careful and follow instructions to prevent injury. Make sure your fingers are away from moving parts that may be closing gaps while moving.

2. Fold both sides of the handle bar (see Figure 33).

Figure 33

3. Release the secure lever of the back handle and bring the handle to 90 degrees position. Allow the seat to slide down until it clicks in place (see Figure 34 & Figure 35).

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Figure 36

Figure 35

4. Push the back of the chair forwards until it clicks (see Figure 36).

5. Use one hand to keep the rod upright. Using your other hand, release the front lever (see Figure 37). In case the front lever require too much force to operate, you should apply horizontal pressure on the main handle towards the front of the scooter. This will release the pressure on the front lever and will make it easier to open.

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Figure 37

Switch hands. Use one hand to keep the rod upright and use your other hand to lift the main handle (see Figure 38).

Figure 38

- Using the main handle pull the center upwards and towards the rod until it clicks and locks in place (see Figure 39).

PINCH POINT Be careful not to pinch your finger between the chassis. This area is a pinch point and requires your full attention.

Figure 39

- Use the rod lever to adjust the height of the handle bar to the lowest position and bring the lever back to the secure position (see Figure 40).

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Figure 40

- Release the adjustment lever of the handle bar and bring the handle bar to a horizontal position. Then bring the lever back to secure position (see Figure 41).

Figure 41

Separating the chassis

⚠ WARNING Lifting ATTO by yourself may be challenging for some people and therefore not recommended. If you have another person to assist you, we suggest lifting the product together. If you need to lift ATTO alone, you can separate ATTO into two segments as described below. Lifting ATTO as a whole may be too heavy and may cause injury or damage.

ATTO's front and rear halves (chassis) can be easily and effortlessly separated in only a few seconds by operating a knob and two handles. The same technique applies to putting the two halves back together. This is very useful when storing ATTO in the trunk or in an airplane's overhead bins.

- Make sure the handle bar is secured at an upright position (see Figure 42).

Figur 42–44

Figure 42

2. Open both separation levers (see Figure 43). In case the separation levers require too much force to operate, you should apply horizontal pressure on the driving console towards the center of the scooter. This will release the pressure on the separation levers and will make them easier to open.

Figure 43

3. Hold the handle bar with one hand and press the central release button with the second hand until it clicks (see Figure 44).

Figure 44

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4. Carry the front chassis from the handle bar and separate from the back chassis (see Figure 45).

Figure 45

- (1) Front handle
- (2) Driving console handle
- (3) Frame (underneath)
- (4) Support leg

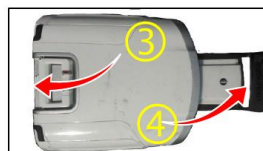
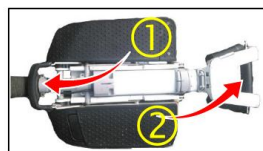
Figure 46 – lifting points

Lifting separated chassis

1. Once the two chassis has been separated and you are ready to lift them, for instance into your car trunk, make sure both chassis is set flat on the ground where it will not be an obstacles to you or to others,
2. Grab the one of the chassis from designated handles and carry it to its destination (see Figure 46):

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3. Repeat the previous step on the other chassis.
4. Lay the chassis flat and secure them with straps (or other ways) to insure safety and damage free transportation.

Connecting the chassis

1. Make sure the handle bar is secured at an upright position (see Figure 47).

Figure 47

Place the back chassis vertically. Hold the front chassis from the handle bar and bring it towards the back chassis (see Figure 48).

Figure 48

2. Bring the two parts together. Use the hanging hooks (see Figure 49) to locate the two ends properly. At this point the central release button will click.

Figure 49

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- Close the two separation levers until they click (see Figure 50). In case the separation levers require too much force to operate, you should apply horizontal pressure on the driving console handle towards the center of the scooter. This will release the pressure on the separation levers and will make them easier to open.

NOTE: DO NOT try to open ATTO for driving before the two levers are secured in the downwards position

Figure 50

Freewheel

ATTO is equipped with a manual freewheel lever, which when pushed to "N" state it in freewheel mode and allows the scooter to be pushed.

WARNING When ATTO is in freewheel mode, the braking system is disengaged, and ATTO will move freely with very little force. This can cause injury and property damage to anyone and anything nearby.

- Change to freewheel mode only on a level surface.
- Ensure ATTO's power is turned off.
- Never sit on ATTO while handling the freewheel lever.

WARNING After you have finished pushing your scooter, always lock the brakes by changing to non-freewheel mode.

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Operating the freewheel

In case the battery dies or ATTO is not driving, ATTO may be rolled manually. This will require releasing the mechanic auto-brake of the front wheel using the free wheel lever (see Figure 51).

Figure 51 Releasing the freewheel

Before driving ATTO again, make sure the freewheel lever is pushed backwards (see Figure 52).

Figure 52 Securing the freewheel

Battery and charger

ATTO's charger is a small yet smart and powerful charger with the ability to connect to any electrical grid rating 110VAC – 240VAC. It is fully approved and safe to use.

- When connecting the charger, connect the DC charging cord to the battery charge plug before connecting the charger AC cord to the electrical outlet.
- When disconnecting the charger, always do so in reverse order of the connecting procedure.

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Figure 54 ATTO's Charger

Installing the battery

To install the battery, insert first the back end of the battery and then push the front until the battery clicks and locks in place (see Figure 55).

NOTE Make sure the battery is properly connected before folding ATTO or turning it on.

Figure 55

Removing the battery

Identify the battery release button highlighted in red frame in Figure 56.

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Figure 56 - battery release button

Figure 58

To remove the battery from ATTO, press the release button and pull the battery out (see Figure 57).

Figure 57

Use the handle for carrying the battery when removed from ATTO (see Figure 58).

⚠ WARNING

- NEVER attempt to recharge the batteries by attaching cables directly to the battery terminals.
- DO NOT attempt to recharge the battery and operate ATTO at the same time.
- DO NOT attempt to recharge the battery when ATTO has been exposed to any type of moisture.
- DO NOT attempt to recharge the battery when ATTO is outside.
- DO NOT sit on ATTO while recharging the battery.

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⚠ WARNING

- New battery MUST be fully charged prior to initial use of ATTO. ALWAYS fully charge new battery before initial use or battery life will be reduced.
- As a general rule, you should charge your battery at 60% of full charge before placing in storage for long period of time.
- Only use a charger approved by Moving Life.
- Use of improper extension cord for the charger could result in risk of fire and electric shock.

Charging the battery can be done on board and off board, meaning you can connect the charger when the battery is in its place inside ATTO or when it is taken out of ATTO.

Charging the battery

📌 NOTE

- a. The battery should not be emptied. To maintain good battery state and maximize its

life span always charge the battery when the battery meter reaches 2 green lines.

- b. Before placing the battery for a long period of time make sure to charge it at about 60% of full charge.

1. Connect the charger to the electric socket. The Green LED should turn green to indicate the charger is properly connected to the socket.
2. Connect the other end of the charger to ATTO's battery. The charger LED will turn red to indicate the battery is not fully charged and that charging is taking place.
3. When the battery is fully charged, the LED will turn green. This may take up to 6 hours. Once charging is complete, disconnect the charger from the wall socket and then disconnect the battery from the charger.

📌 NOTE

Battery can be charged either when connected to ATTO or out of ATTO.

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Maximizing battery's life time

⚠ WARNING Always keep batteries in cool dry conditions to prevent damage to the battery and surrounding equipment.

Installing and using the pelvic support

A pelvic support is an optional device you can purchase separately from Moving Life and install on your ATTO according to following instructions:

1. Place the two Velcro Stickers on the back of ATTO near the back handle as shown in Figure 59. Pill the protecting tape and stick the Velcro in place using the self-adhering side. Apply light pressure to ensure good contact.

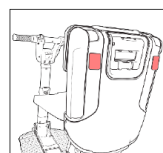
Figure 59 - Placing Velcro Stickers

2. Attach the pelvic support belt to the Velcro stickers as shown in Figure 60. You can use any of the two parts on any side (left or right).

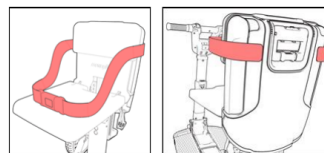
Figure 60 - Attaching the safety belt to the Velcro stickers

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TRANSPORTING ATTO

ATTO can be effortlessly transported in many types of vehicles, on land, see or by air.

Transport of ATTO on commercial aircraft is subject to federal regulations and airline policy. Please consult your airline prior to your flight to ensure your ATTO and its lithium-ion battery is permitted.

To transport ATTO:

- For safety reasons, remove the battery and place it in a safe location.
- Make sure it is fully closed.
- To prevent injury and damage, secure ATTO in a way that will prevent it from moving around or shaking.
- Cover or otherwise secure the tiller and handlebar to prevent it from being damaged due to moving

objects or by ATTO shifting around during transport.

To transport ATTO in a car please see the "Separating the chassis" chapter in this Owner's Manual.

WARNING

- If your scooter and its components are not properly and securely stowed, the scooter or components may shift around while driving causing injury or damage.
- If the scooter is transported in a vehicle when fully assembled and in open position, no one should sit on the scooter during loading and transport.
- If you use an external lifting system, be sure to secure your ATTO as it could fall off the lift.
-

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- It may be necessary to have the charger tested.

Note: ATTO's battery can be charged while it is installed on the scooter or when it is removed from the scooter.

WARNING Do not leave the battery pack and charger plugged in for more than 12 hours. Damage to the battery pack and a hazardous condition may result.

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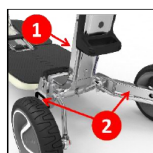
Figure 66

All of the bearings on your ATTO are permanently lubricated and sealed and therefore no lubrication is required.

Battery

WARNING ATTO's battery holds a substantial amount of power. Shorting of the terminals can result in burns or other injuries and in damage to the battery and to the electronic components of your scooter. Always use care and caution when handling the battery.

Battery & Charging

- These batteries are maintenance-free.
- Your scooter is equipped with Li-ion battery, which does not have a "memory" issue and therefore it is not necessary to completely discharge the battery before recharging it.
- If you are not going to use your ATTO for an extended period of time, charge the batteries to 60% of full charge before storing your scooter. The batteries can be left unmaintained for up to 6 months if they are properly charged and kept at room temperature approximately 22 °C (~ 72° F).
- If you have not used your scooter for more than one month, make sure that you charge the batteries on the night before you are going to use your scooter. This will ensure that you start operating your scooter with a fully charged battery.
- There is no danger of spillage or leakage, so these batteries are safe to transport on aircrafts, busses, trains, etc.
- Following the procedures set out in this Owner's Manual will extend the life of your batteries.

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Moving Life Ltd. 2 Haofe St. Kadima 6092000 Israel
Obelis Group Ltd. Bd Général Wahis 53 B-1030 Bryssel Belgien